


IBEX M2 Quick Guide

1. Equipment Configuration

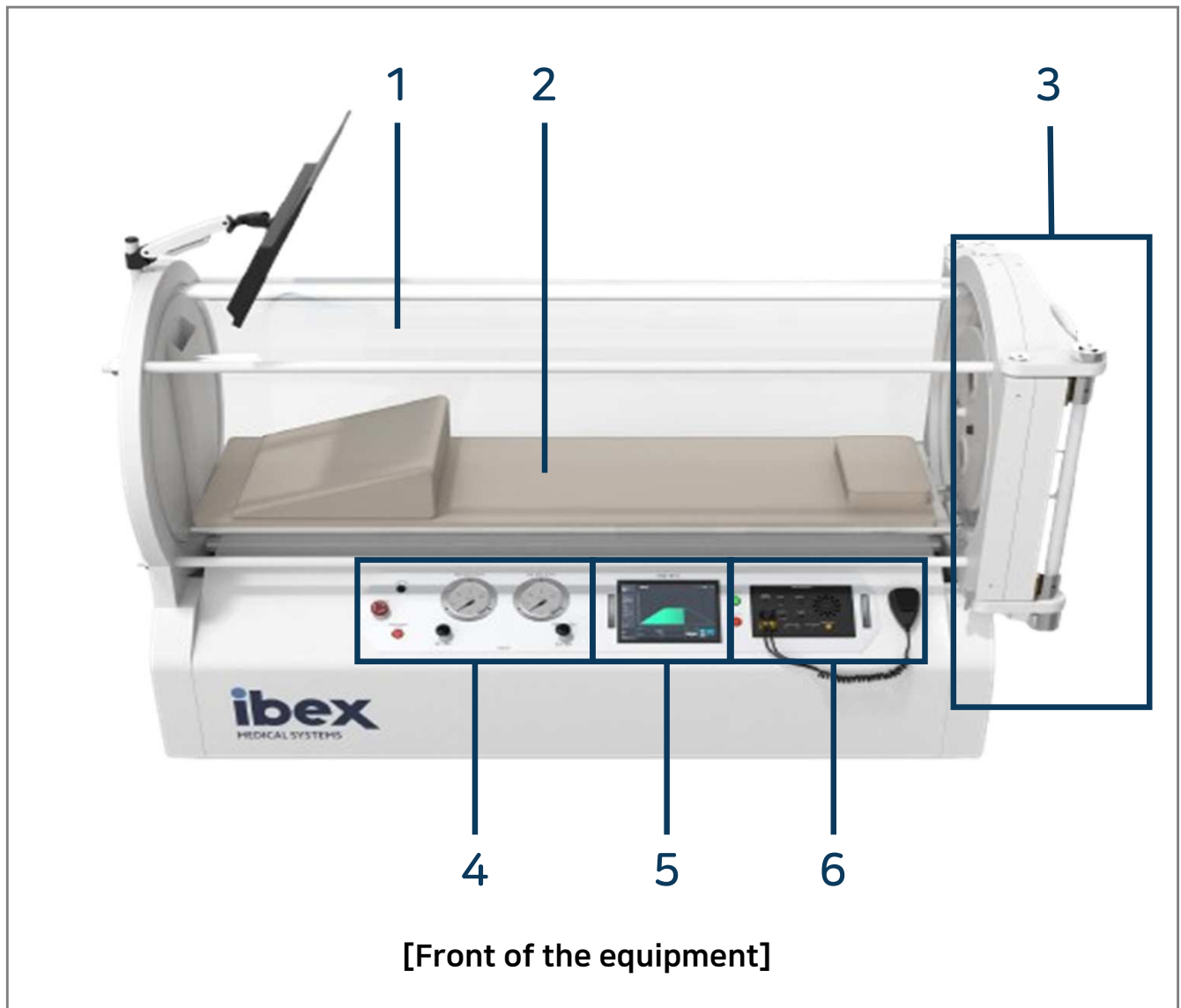

No.	Designations	Function
1	Acrylic cylinder	observation of both the user and the patient
2	Patient stretcher	the patient to receive treatment comfortably while lying down
3	Chamber Door and Locking Device	Patient Entrance and Door Locking Device
4	Manual control panel	Panel for manual operation of the chamber
5	Auto control panel	Panel for automatic operation of the chamber
6	Communication device	A device for communicating with the patient


2. Pre-use Checklist

Emergency Exhaust Valve  Demand Valve Exhaust Valve	• Ensure that the supply valve and exhaust valve are completely closed. ✘ If the Demand valve is open, an excessive amount of oxygen be supplied, causing the pressure to rise rapidly beyond the set pressurization level, which may put a strain on the patient. ✘ If the exhaust valve is open, oxygen continue to be vented, preventing the chamber from reaching the set pressurization level. • Check the emergency exhaust valve is released. ✘ If the emergency exhaust valve is pressed, oxygen continue to be vented, preventing the chamber from reaching the set pressurization level.
	• Make sure the UPS power is turned on. - If the UPS power is off, press the power button to keep it turned on.
	• the chamber's main power cord is connected, the UPS will automatically charge. • However, if the chamber's main power cord is not connected and the UPS power is turned on, the UPS battery may be drained and discharged. • Therefore, when disconnecting the chamber's main power cord, <u>you must turn off the UPS power as well.</u>


3. How to Use




Turn on the chamber power.

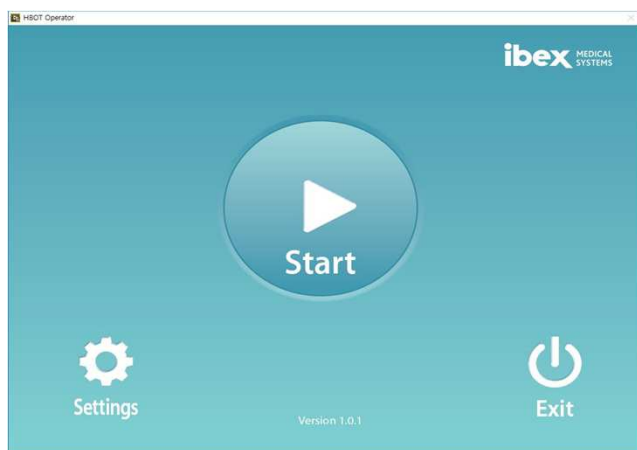


After pressing the Start button, proceed with the login.

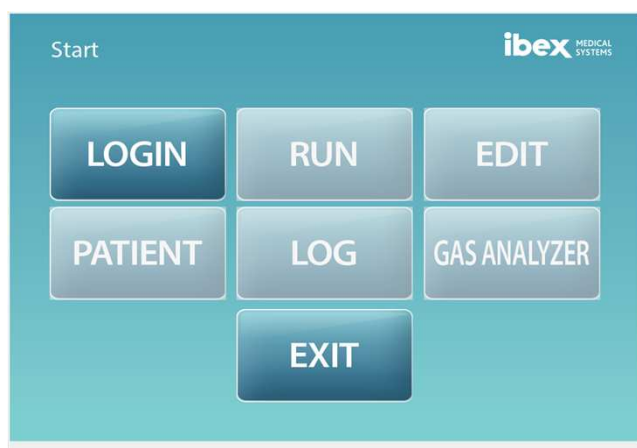


3. How to Use

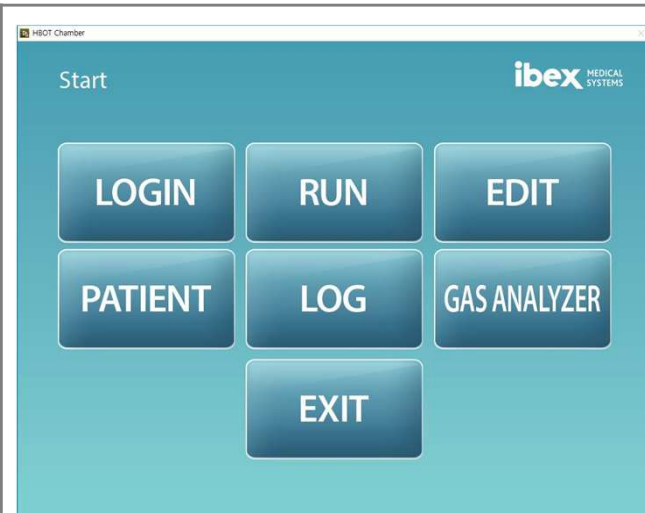

- Press the RUN button.

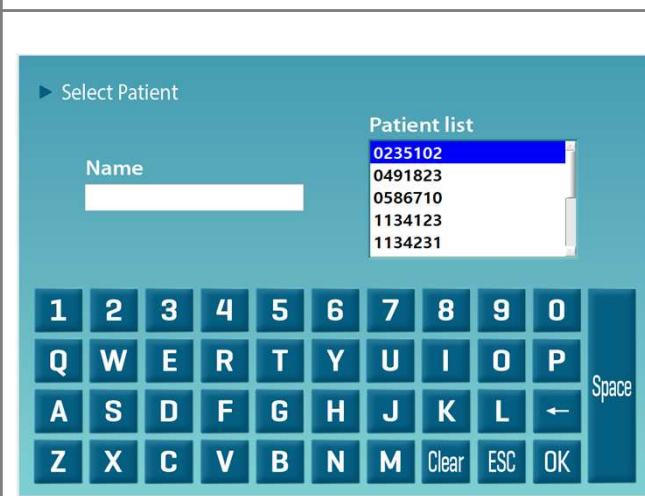

- Select the patient list for treatment.

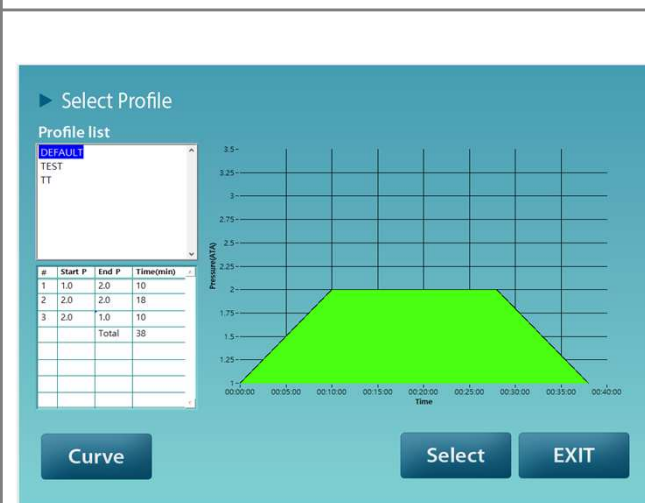

- Select the treatment program want to proceed with and complete all the necessary preparations.

3. How to Use



- Secure the stretcher to the chamber.



- Move the patient into the chamber using the stretcher.



- Press the RUN button to start the treatment.

4. How to Communication Device



1. Turn on both the communication device power and the speaker power.
2. Adjust the operator volume (chamber operator's voice) and chamber volume (patient's voice) appropriately.
3. Press the button on the left side of the speaker to communicate with the patient.